

flexeal

Affixing


1. Take the *flexeal* strip out of the packaging and peel off the protective foil. It is best to do this from the side with the overhang of the protective film.



2. Place the face mask with the outside facing upwards. The transparent strip on the face mask is at the top edge of the outside of the mask.



3. Press the *flexeal* strip lightly by hand onto the backing foil of the face mask. The *flexeal* strip protrudes approx. 1 cm over the edge of the face mask.

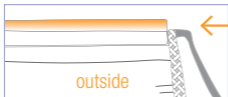


4. Put the face mask to your face and apply some pressure, starting in the middle and moving from the nose to the left and right. Only when the face mask fits tightly from the nose to the eye area, attach the mask to the ears with the rubber loops.



flexeal

To remove the mask, pull off the *flexeal* strip from the side towards the nose. If the face mask is not worn, there is a carrier foil on the inside of the face mask to protect the *flexeal* strip. Simply press the *flexeal* strip around the upper edge to the inside of the face mask.

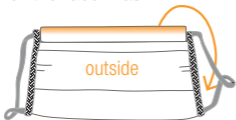

The procedure for the use with an own face mask is the same as above. The *flexeal* strip also sticks to common face masks that do not provide a special transparent strip. In this case, if the face mask is not worn, press the backing foil back onto the *flexeal* strip. The *flexeal* strip does not need to be removed from the face mask.

Cleaning

Detach the *flexeal* strip from the mask with a gentle pull from the side. Wash the sticky side of the *flexeal* strip under running water with detergent. For optimal cleaning, the *flexeal* strip can also be cleaned with conventional disinfectant. Allow to air dry.

After the cleaning process, the *flexeal* strip can be reapplied to the mask as described.

The face mask can be washed at 60° C.